Fig. 1.

Fig. 2.

Fig. 3.

Fig. 4.

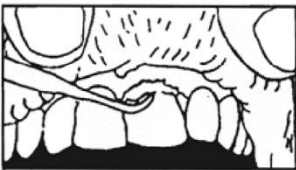
Fig. 5.

ENGLISH

FIRST AID KIT FOR TEETH **INSTRUCTIONS**

READ CAREFULLY ALL INSTRUCTIONS BEFORE USING THE KIT.

The Dentanurse kit is designed for first aid only and SHOULD NOT be used in cases of prolonged pain or where there is facial swelling. Prompt first aid however, will often prevent later complications, but professional treatment must be sought as soon as possible after first aid. It must be stressed that to prevent cross infection ONE kit should be used for ONE patient. However the same kit can be used several times on the same patient.

PROFESSIONAL HELP SHOULD BE OBTAINED AS SOON AS POSSIBLE, BUT IN AN EMERGENCY THE PROCEDURE MAY BE REPEATED IF NECESSARY.

CONTENTS

Dental mirror, application spatula, multi-tool, base & catalyst paste, mixing tray, instructions for use.

HOW TO USE THE MIRROR

To obtain an angle, place the head of the mirror on a flat surface with the reflecting surface uppermost. Bend the handle forwards until it is flat against the reflector. This puts a crease in the neck of the mirror. The angle can be easily adjusted by the user (Fig. 1)

MODE OF USE: (Fig. 2). The mirror is designed to be used in conjunction with another mirror eg hand or wall mirror. Adjust the angle to get the best results. A good source of light is essential. The demisting quality of the mirror can be enhanced by immersing the mirror head in warm water just before use.

REPLACING A LOST FILLING OR BROKEN TOOTH

1. WASH HANDS THOROUGHLY.
2. Rinse the mouth free of food.
3. Use the Dentanurse mirror to visualise the interior of the mouth.
4. Using the Dentanurse multi-purpose instrument's curved end, gently clear any particles from the cavity, (Fig. 3). It is essential that no food be trapped in the cavity when inserting the Dentanurse filling.
5. Remove the caps from the tubes of Dentanurse base and Dentanurse catalyst, and using the top end of each cap, pierce the metal seals. It is important that the punctured holes are of equal diameter to ensure correct paste mix.

6. Measure equal amounts (2cms approx) each of base and catalyst onto the mixing tray, mixing both together thoroughly for ten seconds.
7. Gather the cement mixture onto the end of the spatula and allow to stand until the cement becomes stiff enough to roll between a thumb and finger. Place into the cavity and press firmly with the plugger end of the multi-purpose tool (fig 4). The cavity should be filled to the rim and then the teeth closed together for five minutes. After five minutes, excess material can be trimmed from around the cavity using the curved end of the multi-purpose tool. Allow two hours after repair before eating.

REPLACING DISLODGED INLAYS

Dislodged inlays are best treated as lost fillings. The inlay should be put in a safe place until professional help is obtained, but the inlay cavity should be dressed with Dentanurse base/catalyst mixture as for a lost filling. Allow two hours after repair before eating.

REPLACING DISLODGED CROWNS AND BRIDGES

1. WASH HANDS THOROUGHLY
2. Rinse the mouth free of food debris-use a toothbrush if available.
3. Use the Dentanurse mirror to visualise the interior of the mouth. Use the Dentanurse multi-purpose instrument's curved end to gently clear away any particles from around the tooth/teeth. This end should also be used to remove any old cement from the fitting surface of the dislodged crown or bridge.
4. Re-seat the bridge or crown back into the mouth. The patient will know when the bridge or crown is correctly seated, as they can close their teeth back together again comfortably. When satisfied that the seating is correct, remove the crown or bridge again.
5. Remove the caps from the tubes of Dentanurse base and catalyst and using the top end of each cap, pierce the metal seals. It is important that the punctured holes are of equal diameter to ensure correct paste mix.
6. Measure equal amounts (2cms approx) each of base and catalyst onto the mixing tray, mixing both together thoroughly for ten seconds.
7. Using the spatula, transfer the cement to the fitting surface of the crown/bridge, and re-seat in the mouth as quickly as possible.
8. After ten minutes, use the tip of the multi-purpose tool CAREFULLY to remove excess cement from round the margin of the crown/bridge (Fig 5).

CONTRA

Contains eugenol and should not be used on patients with a known allergy to eugenol. May cause allergic reaction. If any such reactions are experienced, discontinue use of product and seek medical assistance. IRRITANT. Avoid contact with skin and eyes.

DENTANURSE CEMENT IS PURELY TEMPORARY AND PROFESSIONAL HELP SHOULD BE OBTAINED AS SOON AS POSSIBLE, BUT IN AN EMERGENCY THE PROCEDURE MAY BE REPEATED. ALLOW TWO HOURS AFTER REPAIR BEFORE EATING. RECOMMENDED USE IS UP TO A MAXIMUM OF 4 WEEKS.

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[REF]


1.000: BASE 10g 1.902/ Catalyst 10g 1.904
1.001: BASE 10g 1.902/ Catalyst 10g 1.904
1.101: BASE 10g 1.902/ Catalyst 10g 1.904
1.100: BASE 10g 1.902/ Catalyst 10g 1.904
1.200: BASE 10g 1.902/ Catalyst 10g 1.904

Tel: +44 1892 870500 Fax: +44 1892 870482
Email: sales@ahl.uk.com

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging.

2021-04 AP10212/1

Advena Ltd.
Tower Business Centre, 2nd Flr.,
Tower Street, Swatar,
BKR 4013 Malta.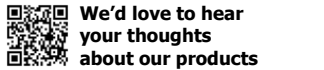

We'd love to hear your thoughts about our products

At Dentanurse we're committed to continuous improvement and regulatory compliance. So, if you've purchased any of our products we'd really appreciate five minutes of your time to complete a short survey. Thank you in advance, your feedback and comments are truly valued by us. Please scan the QR code or go to **https://forms.gle/o6PxxksNnfu7NdRq7**